


Clip protection from dislodgement means **in-jaw security**

Endoscopic Multiple Clip Applier Fact Sheet

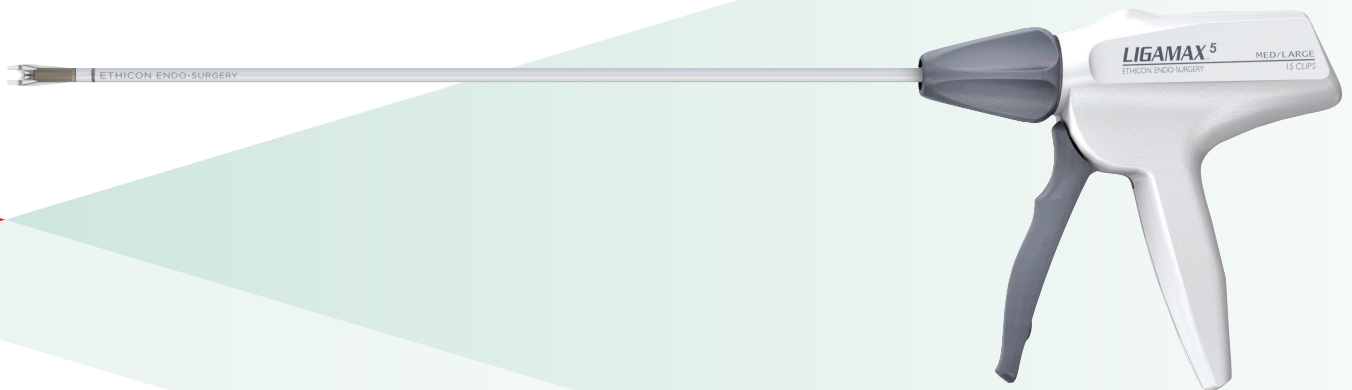

LIGAMAX5 Features

Collapsible Jaws

Allows jaws to collapse down and fit in 5mm trocar

Trocar Insertion Line

Indicator that the instrument is inserted far enough in the trocar for jaws to open fully

Clip Indicator

Orange bar appears after the 13th clip has been fired and the window progressively fills to indicate that 2, 1, or no clips remain

Tissue Stop

Minimizes contact of the vessel or duct with the apex of the clip to minimize the chance of clip dislodgement

Clip Advancer Tab

Facilitates clip feeding and provides contact point during formation to aid stability during the firing cycle

Anti-backup Ratchet Mechanism

Prevents interruptions in the firing sequence from affecting in-jaw clip stability

Clips Designed to Hold Means On-structure Security

Tongue and Groove

Provides leg alignment as the two clip legs interlock once the clip is closed, minimizing closed clip gap

Transverse and Lateral Grooves

Provide gripping surface to aid on-structure clip retention compared to smooth clip

Apex Notch

Reduces the force to close the clip

Crimp Pads

Provide points of contact during the final portion of firing to minimize closed clip gap at the proximal end of the clip

Product Overview

- Three-point contact – Jaws and the clip advancer tab hold the clip in place during clip loading, surgical maneuvering and firing to minimize the chance of clip dislodgment
- Protection at the clip apex is provided by the tissue stop as the device is positioned, minimizing the chance for clip dislodgment
- Crimp pads, apex notch and tongue and groove aid in proper occlusion along the full length of the clip – proximal to distal
- Axial and lateral clip grooves minimize the risk of clip migration, increasing the force to remove the clip from a structure compared to a smooth clip

Ordering Information

Product Code	Description	Clip Size	Closed clip length	Number of clips	Qty/box
EL5ML	LIGAMAX5 Endoscopic Multiple Clip Applier	Medium/Large	8.8mm	15	3

How to Order

All purchase orders are made to Johnson & Johnson Health Care Systems, Inc. (JJHCS) or your authorized distributor.

Electronic Ordering Options

(Note: Placing orders electronically avoids minimum order fees for your hospital.)

Johnson & Johnson Gateway:
www.jnjgateway.com/commerce

For questions about your order, visit the website or call 1-866-JNJ-GATE (565-4283).

Global Healthcare Exchange (GHX):
www.ghx.com

For questions about your order, visit the website or call 1-800-YOUR-GHX (968-7449).

Electronic Data Interchange (EDI):
JJHCS EDI Help Line: 1-800-262-2888

Nonelectronic/Manual Ordering Options

Call JJHCS at 1-800-255-2500 between 8:30 a.m. and 9:00 p.m. Eastern Standard Time or fax your order to 1-732-562-2212.

Customer support

For product use assistance, clinical guidelines, service and repair, emergency assistance, copy of a 510(k) clearance letter or complaints, please contact our Customer Support Center at customersupport@eesus.jnj.com or by calling **1-877-ETHICON** (384-4266). Our Support Center is staffed 24 hours a day, 7 days a week by qualified nurses to answer your product-related questions.