

UNIUM™ System

Care and Maintenance

Disassembly | **1a Manual Cleaning** | **1b Automated Cleaning** | **2 Lubricating** | **3 Check Function** | **4 Sterilization**


Disassembly	1a Manual Cleaning Manual Cleaning Disinfection	1b Automated Cleaning Manual Pre-Cleaning	Automated Cleaning Cleaning and Disinfection	2 Lubricating	3 Check Function	4 Sterilization
 05.001.601  05.001.611  05.001.603	 2-3X  2-3X <p>519.400/519.40** 532.024</p>	 2-3X  2-3X <p>519.400/519.40**</p>	 68.001.650 +68.001.654  68.001.651	 2-3X  2-3X <p>519.970/ 519.97**</p>	 	 68.001.650 +68.001.654  68.001.651
 05.001.604  05.001.602  05.001.204	 	 	 	 	 Press 8 Sec.	
<p>Modular Handpiece Attachments</p>  05.001.252  05.001.253  05.001.254  05.001.188  05.001.250  05.001.251  05.001.257, 532.017  05.001.258, 532.018  05.001.259, 532.019  05.001.260, 532.020  532.015  532.011  532.012  05.001.187  532.021  532.023  532.026  511.300, 532.30**  532.031  310.932/ 310.93K**  510.191  532.027 <p>Reciprocating Saw Handpiece Accessory</p>  05.001.612	 *  *  *  *  *  * <p>519.400/519.40** (for attachment 532.023 use brush 532.024)</p>  *  *  *  *  *  *  *  *  *  *  *  *  *  *  *  *  *	 *  *  *  *  *  * <p>519.400/519.40** (for attachment 532.023 use brush 532.024)</p>  *  *  *  *  *  *  *  *  *  *  *  *  *  *  *  *  *	 68.001.650 +68.001.654  68.001.650 +68.001.654  68.001.650 +68.001.654  68.001.650 +68.001.654  68.001.650 +68.001.654  68.001.650 +68.001.654  68.001.650 +68.001.654  68.001.650 +68.001.654  68.001.650 +68.001.654  68.001.650 +68.001.654 68.001.650 +68.001.654 68.001.650 +68.001.654 68.001.650 +68.001.654 68.001.650 +68.001.654 68.001.650 +68.001.654 68.001.650 +68.001.654 68.001.650 +68.001.654 68.001.650 +68.001.654 68.001.650 +68.001.654 68.001.650 +68.001.654 68.001.650 +68.001.654 68.001.650 +68.001.654 68.001.650 +68.001.654	 *  *  *  *  *  *  *  *  *  *  *  *  *  *  *  *  *  *  *  *  *  *  *	 *  *  *  *  *  *  *  *  *  *  *  *  *  *  *  *  *  *  *  *  *  *  *	 68.001.650 +68.001.654  68.001.650 +68.001.654  68.001.650 +68.001.654  68.001.650 +68.001.654  68.001.650 +68.001.654  68.001.650 +68.001.654  68.001.650 +68.001.654  68.001.650 +68.001.654  68.001.650 +68.001.654  68.001.650 +68.001.654 68.001.650 +68.001.654 68.001.650 +68.001.654 68.001.650 +68.001.654 68.001.650 +68.001.654 68.001.650 +68.001.654 68.001.650 +68.001.654 68.001.650 +68.001.654 68.001.650 +68.001.654 68.001.650 +68.001.654 68.001.650 +68.001.654 68.001.650 +68.001.654 68.001.650 +68.001.654 68.001.650 +68.001.654

** available in USA only

PICTOGRAM KEY

Explanation of the pictograms used in the poster

Note: Symbols shown at the top of each Care and Maintenance step in the poster apply to all article groups, if not indicated differently.

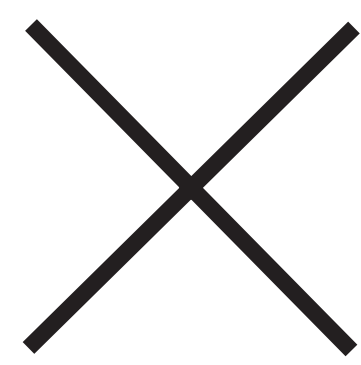


When a symbol in the poster is marked with a cross, the action is forbidden. Not following this may result in damage to the product and patient or user harm.



article#

Caution: Use the mentioned article for this step

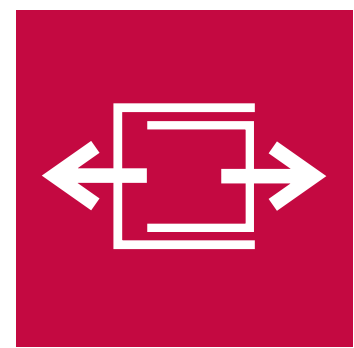

Disassembling




Spray with disinfectant and wipe off*



Immersion in disinfectant possible*



Wipe off with an absorbent cloth soaked in disinfectant*

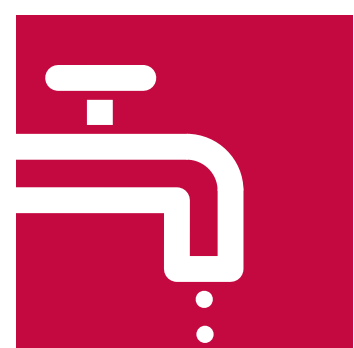

Manual cleaning



Automated cleaning with manual pre-cleaning



Immersion allowed



Immersion and automated cleaning allowed



Check the function of parts marked in red



Lubricate the designated points. See product-specific "Instructions for Use" for more detailed information



Steam sterilization



Hot air, ethylene oxide, plasma and formaldehyde sterilization



H₂O₂ plasma STERRAD sterilization (max. 60 °C/140 °F)



H₂O₂ V-PRO sterilization (max. 50 °C/122 °F)



Do sterilize!

Please refer to the instructions for use of UNIUM and Universal Battery Charger II for a complete list of indications, contraindications, warnings and precautions.

All Synthes implant instructions for use as well as other instructions for use are available as PDF files at www.e-ifu.com.

Not all products are currently available in all markets.

Please contact your DePuy Synthes sales representative for more information.

* Follow disinfectant manufacturer's instruction.

** Refer to product label for legal manufacturer.



Synthes GmbH
Eimattstrasse 3
4436 Oberdorf
Switzerland
Tel: +41 61 965 61 11



Synthes GmbH
Luzernerstrasse 21
4528 Zuchwil
Switzerland
Tel: +41 32 720 40 60



Synthes USA, LLC
1302 Wrights Lane East
West Chester, PA 19380
USA
Tel: +1 640 719 5000



Synthes USA, LLC
1101 Synthes Avenue
Monument, CO 80132
USA
Tel: +1 719 481 5300