


# Loading Plan of Washing and Sterilization Basket, size ½, for UNIUM™ Reciprocating Saw Handpiece (68.001.651)

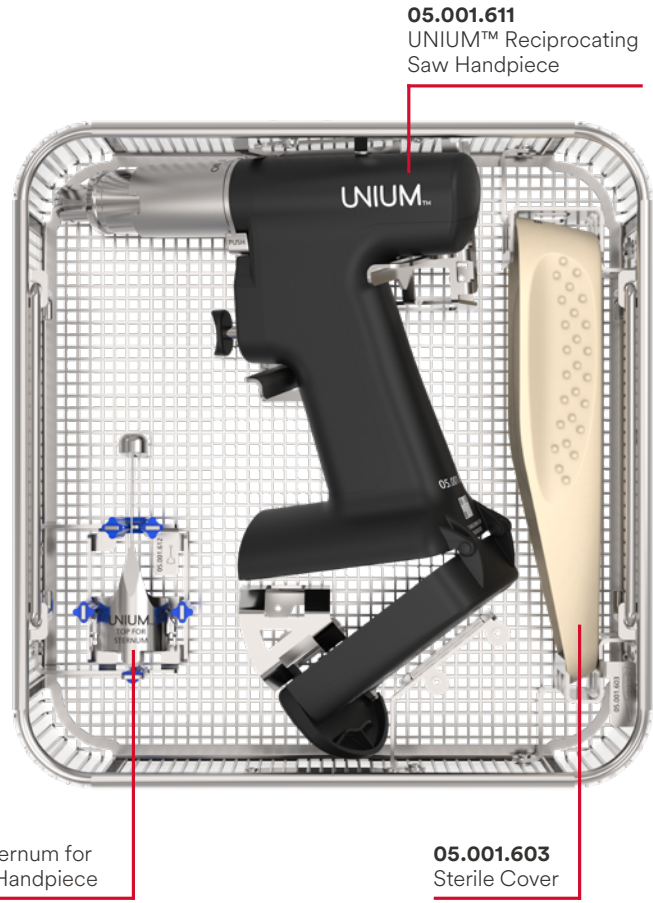

**05.001.611**  
UNIUM™ Reciprocating  
Saw Handpiece

**05.001.612**  
UNIUM™ Top for Sternum for  
Reciprocating Saw Handpiece

**05.001.603**  
Sterile Cover


**Dimensions  
(Length × Width × Height)**

Basket without Lid:  
240 × 250 × 100 mm  
  
Basket with Lid:  
240 × 251 × 108 mm

**Weight**

Washing Basket (68.001.651):  
1.08 kg/2.38 lbs  
  
Lid (68.001.652):  
0.4 kg/0.88 lbs  
  
Total weight when empty:  
1.48 kg/3.26 lbs

## Lid for Washing and Sterilization Basket, size ½, for UNIUM™ (68.001.652)

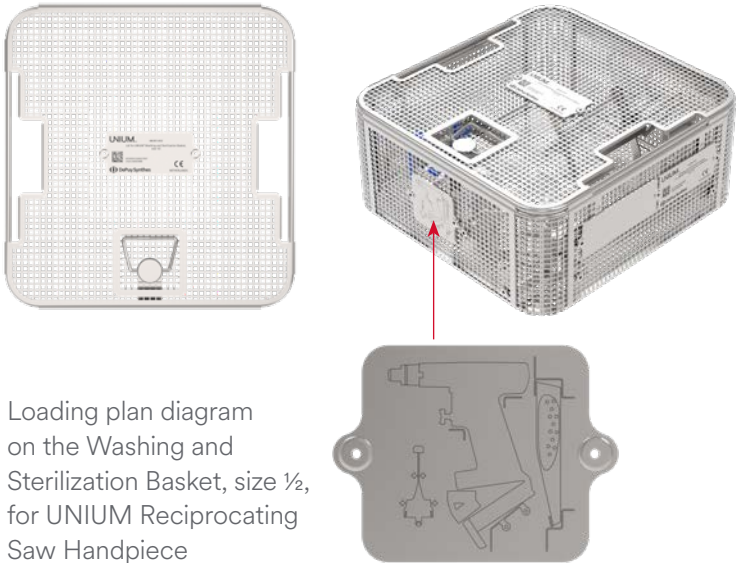

Loading plan diagram  
on the Washing and  
Sterilization Basket, size ½,  
for UNIUM Reciprocating  
Saw Handpiece

**Notes:**

- Ensure that the casing cover of the UNIUM Modular or Reciprocating Saw Handpiece is open and the front lid is turned 180° until it clicks. Handle the front lid, casing cover, edges, seals and hinges with care.
- Alternatively, the UNIUM Modular Handpiece 05.001.601 can also be placed in the Washing and Sterilization Basket, size ½ (68.001.651).
- The images show the basket fully loaded, all items must be ordered separately.



For detailed information please refer to the instructions for use for a complete list of indications, contraindications, warnings and precautions. Not all products are currently available in all markets. All DePuy Synthes implant instructions for use as well as other instructions for use are available as PDF files at [www.e-ifu.com](http://www.e-ifu.com). Technical data is subject to tolerances. Please contact your DePuy Synthes sales representative for more information.

\* Refer to product label for legal manufacturer  
[www.depuyssynthes.com](http://www.depuyssynthes.com)

**Synthes GmbH**  
Eimattstrasse 3  
4436 Oberdorf  
Switzerland  
Tel: +41 61 965 61 11

**Synthes GmbH**  
Luzernerstrasse 21  
4528 Zuchwil  
Switzerland  
Tel: +41 32 720 40 60

**Synthes USA, LLC**  
1302 Wrights Lane East  
West Chester, PA 19380  
USA  
Tel: +1 640 719 5000

**Synthes USA, LLC**  
1101 Synthes Avenue  
Monument, CO 80132  
USA  
Tel: +1 719 481 5300