

Attune Knee System Revision LPS Insert AOX

Product Codes: 151760112, 151760114, 151760116, 151760118, 151760120, 151760122, 151760124, 151760212, 151760214, 151760216, 151760218, 151760220, 151760222, 151760224, 151760226, 151760229, 151760232, 151760312, 151760314, 151760316, 151760318, 151760320, 151760322, 151760324, 151760326, 151760329, 151760332, 151760612, 151760614, 151760616, 151760618, 151760620, 151760622, 151760624, 151760626, 151760629, 151760632

Consumer Medical Device Information

What is in this leaflet

This leaflet answers some common questions about Attune Knee System Revision LPS Insert AOX. It does not contain all the available information. It does not take the place of talking to your surgeon.

All medical devices and implants have risks and benefits. Your surgeon has weighed the risks of using Attune Knee System Revision LPS Insert AOX against the benefits that are expected. This leaflet does not contain all the available information about Attune Knee System Revision LPS Insert AOX. Your surgeon has been provided additional information and can answer any questions you may have. Follow your surgeon's advice even if it differs from what is in this leaflet.

Please read this leaflet carefully and keep it in a safe place so you may refer to it in future if needed.

What is Attune Knee System Revision LPS Insert AOX?

The knee joint consists of three bones: the thigh bone (femur), shin bone (tibia) and kneecap (patella). Knee joint replacement surgery involves removing the worn and damaged areas of the knee joint and replacing them with artificial implants. Attune Knee System Revision LPS Insert AOX is placed between the femur and tibia implants and provides a smooth surface on which the femur implant moves. It is made from polyethylene (a medical-grade plastic) and a metal alloy called cobalt chrome molybdenum.

What is Attune Knee System Revision LPS Insert AOX used for?

Attune Knee System Revision LPS Insert AOX is used for patients with conditions such as non-inflammatory degenerative joint disease (e.g. osteoarthritis), inflammatory joint

disease (e.g. rheumatoid arthritis), malignant tumours, failed previous implant, severe trauma or post-infection bone loss that require extensive removal and replacement of the knee joint. Replacement of the damaged knee joint aims to relieve pain, increase range of motion, provide a stable joint or correct a deformity to improve the ability to manage daily activities and improve quality of life.

Attune Knee System Revision LPS Insert AOX can only be implanted surgically by a qualified surgeon. Your surgeon will choose the knee replacement for you based on durability, level of performance, wear resistance, their experience or preference and your personal needs. As with any medical treatment, individual results may vary.

When should Attune Knee System Revision LPS Insert AOX not be used?

Attune Knee System Revision LPS Insert AOX should not be used in patients with:

- An infection
- Loss of bone or musculature, osteoporosis, neuromuscular compromise or vascular deficiency in the affected limb

If you are unsure whether Attune Knee System revision LPS Insert AOX should be used in your treatment, talk to your surgeon.

What to do after Attune Knee System Revision LPS Insert AOX has been implanted?

Having a knee joint replaced is a major operation. While most people have a good result and an active recovery, there are potential surgical and medical risks and recovery takes time. Ask your surgeon for more details and talk to them if you have any unusual symptoms such as:

- Pain, redness and swelling in the lower leg
- Suddenly becoming short of breath, chest pain and a cough
- Allergic reaction
- Infection of the surgical site
- Ongoing pain in the knee joint

The performance of Attune Knee System Revision LPS Insert AOX is dependent on factors such as age, physical condition, weight and activity level. Factors such as weight, excessive physical activity or trauma to the area may affect the function and lifespan of the knee joint replacement. It is designed for activities of daily living, not high impact sports. Walking, swimming and cycling are recommended. High impact sports such as jogging or running, playing tennis or repeated heavy lifting can affect the function and lifespan of your new joint. However, ongoing physiotherapy and exercise are important for recovery so speak to your surgeon or physiotherapist about rehabilitation exercises and your lifestyle and specific activity goals.

Magnetic Resonance Imaging (MRI) Safety Information

Speak to your surgeon about whether you can have MRI scans. You should inform the technicians performing the scan that you have Attune Knee System Revision LPS Insert AOX implanted.

Reporting adverse effects

If you wish to report any adverse effects you believe are a result of Attune Knee System Revision LPS Insert AOX, please talk with your surgeon or report the information to Johnson & Johnson Medical Product Safety Department on: productsafetyjjmanz@its.jnj.com

Reports may also be made directly to the Therapeutic Goods Administration via the website:

<https://www.tga.gov.au>

Sponsor

Johnson & Johnson Medical Pty Ltd t/a
DePuy Synthes
1 -5 Khartoum Road
NORTH RYDE, NSW 2113
Tel: 1300 562 711

Revision: 01