


EG1[®] HIGH SPEED ELECTRIC SYSTEM

Care and Maintenance

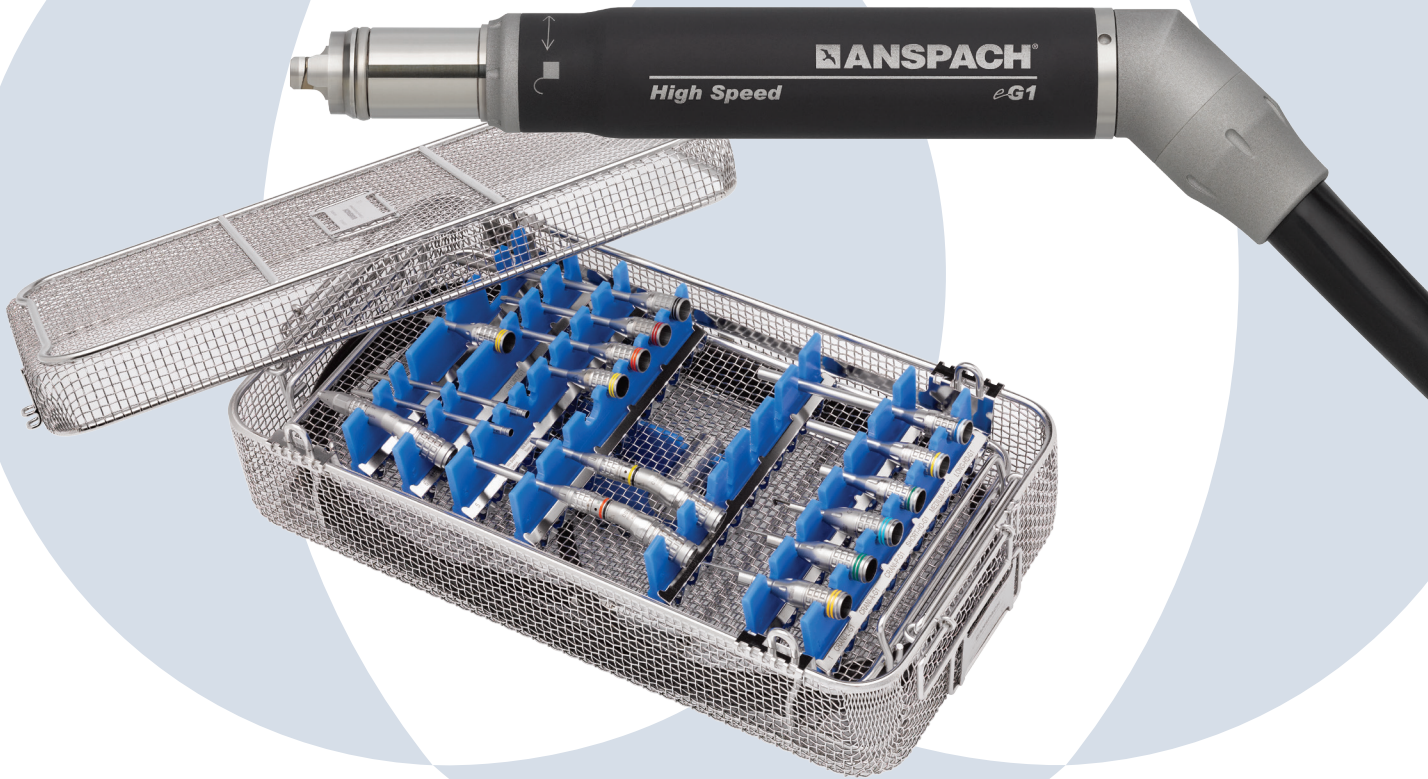

EG1® HIGH SPEED ELECTRIC SYSTEM CARE AND MAINTENANCE



EG1® System	1a Manual Cleaning	1b Automated Cleaning		2 Lubricating	3 Check Function	4 Sterilization
		Manual Pre-Cleaning	Cleaning			
Handpieces						
Foot Controls & Consoles						
Craniotomes						
Straight Attachments						
Angle Attachments						
Minimal Invasive Attachments						
Perforator Driver						

PICTOGRAM KEY



Manual cleaning under running water



Immerse in cleaning solution



Automated cleaning with manual pre-cleaning



Lubricate the designated points. See product-specific "Instructions for Use" for more detailed information. The number of drops is shown on the Care and Maintenance posters.



Flush the inner lumen with a syringe



Steam sterilization



Immerse in instrument milk (non-silicone based medical lubricant)



Wipe off with an absorbent cloth soaked in disinfectant



Check the function of parts marked in red



Do not flush with syringe



Manual cleaning under running water not allowed



Immersing and automated cleaning not allowed



Do Not Lubricate



Immersing not allowed



Immersing not allowed



Steam sterilization only. Hot air, ethylene oxide, plasma and formaldehyde sterilization are not recommended



Do not sterilize!



Caution: Use the mentioned product for this step

Detailed instructions for manual cleaning, automated cleaning and steam sterilization can be found in the product-specific "Instructions for Use"



Manufacturer
The Anspach Effort, Inc.
 4500 Riverside Drive
 Palm Beach Gardens, FL 33410
 USA
 Tel: (800) 327-6887
 Fax: (561) 627-1265
 Email: infoPowerTools@DPYUS.jnj.com

www.depuysynthes.com

EC REP Synthes GmbH
 Eimattstrasse 3
 4436 Oberdorf
 Switzerland
 Tel. +41 61 965 61 11
 Fax +41 61 965 66 00

The third party trademarks used herein are the trademarks of their respective owners.

Limited Warranty and Disclaimer: The Anspach Effort, Inc. products are sold with a limited warranty to the original purchaser against defects in workmanship and materials. Any other express or implied warranties, including warranties of merchantability or fitness, are hereby disclaimed. Please contact Customer Service for full Warranty statement.

WARNING: In the USA, this product has labeling limitations. See package insert for complete information.

CAUTION: USA Law restricts these devices to sale by or on the order of a physician. To order in the US, call DePuy Synthes Power Tools Customer Service: (800) 327-6887.

Not all products are currently available in all markets.