

LUBRICATION INSTRUCTIONS

For battery-driven power equipment



GENERAL LUBRICATION INSTRUCTIONS

For battery-driven power equipment

Small Battery Drive, Trauma Recon System and Battery Power Line

Use only *DePuy Synthes Power Tools* Autoclavable Oil (519.97). Oil drive units/handpieces, battery casings and lids, and attachments after every use. Lubrication should be performed after cleaning and before sterilization.

Apply 1 drop of oil to all accessible moving parts of the drive unit/handpiece, battery casings and lids, and attachments. Distribute the oil by moving the components. Wipe off excess oil with a cloth.

Notes: Attachments and accessories may only be lubricated with *DePuy Synthes Power Tools* Autoclavable Oil. The composition of the vapor-permeable oil is optimized for the specific requirements of the power system. Lubricants with other compositions can cause jamming, have a toxic effect, or have a negative impact on the sterilization results.

To ensure a long service life and to reduce repairs, always lubricate the drive units/handpieces, battery casings and lids, and attachments after each use.

Only lubricate the drive units/handpieces, battery casings and lids, and attachments when they are clean.

DePuy Synthes Power Tools recommends running the drive unit/handpiece with lubricated attachments after cleaning and before sterilization.


1 = 1 Drop

Drive Units/Handpieces



Battery Casings and Lids

Do not lubricate power module (05.001.202).

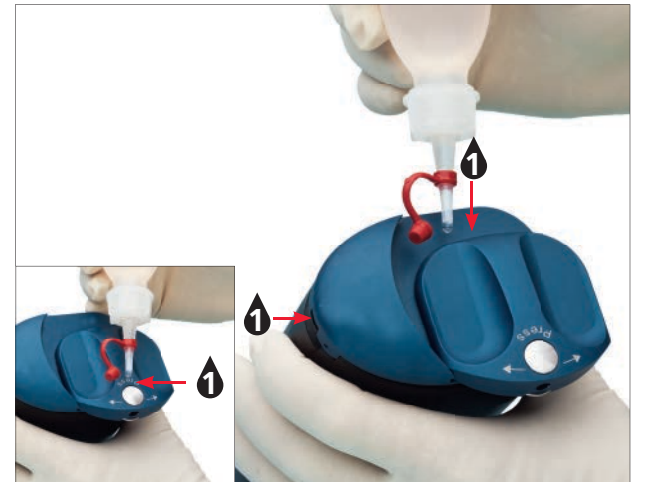

Attachments

Do not lubricate the Radiolucent Drive (511.30).



Limited Warranty and Disclaimer: Products are sold with a limited warranty to the original purchaser against defects in workmanship and materials. Any other express or implied warranties, including warranties of merchantability or fitness, are hereby disclaimed. Please contact Customer Service for full Warranty statement.

WARNING: in the USA, this product has labeling limitations. See package insert for complete information.

CAUTION: USA Law restricts these devices to sale by or on the order of a physician. To order in the US, call DePuy Synthes Power Tools Customer Service: (800) 327-6887.

Not all products are currently available in all markets.

Synthes USA, LLC

1302 Wrights Lane East
West Chester, PA 19380

USA

Telephone: (610) 719-5000

To order: (800) 327-6887

Fax: (800) 327-6661

E-mail: infoPowerTools@DPYUS.jnj.com

www.depuysynthes.com

Synthes (Canada) Ltd.

2566 Meadowpine Boulevard
Mississauga, Ontario L5N 6P9

Telephone: (905) 567-0440

To order: (800) 668-1119

Fax: (905) 567-3185



COMPANIES OF  Johnson & Johnson

© DePuy Synthes Power Tools, a division of DOI 2014. All rights reserved.
DSUS/PWT/0614/0036 6/14 DV