


SURGIFLO™ Hemostatic Matrix Family of Products



SURGIFLO™ Hemostatic Matrix Kit with Thrombin



SURGIFLO™ Hemostatic Matrix

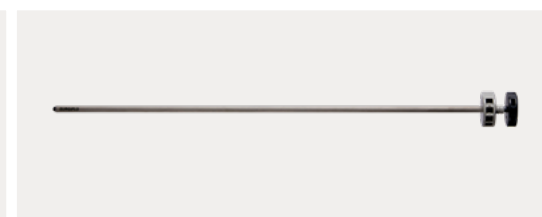

SURGIFLO™ Endoscopic Applicator

	SURGIFLO™ Hemostatic Matrix Kit with Thrombin	SURGIFLO™ Hemostatic Matrix	SURGIFLO™ Endoscopic Applicator
Codes	SF2995, 2994	2991	MS1995
Description	Flowable gelatin matrix with thrombin	Flowable gelatin matrix	Delivery device for minimally invasive procedures
Indication	SURGIFLO™ Hemostatic Matrix, mixed with thrombin solution, is indicated in surgical procedures (other than ophthalmic) as an adjunct to hemostasis when control of bleeding by ligature or other conventional methods is ineffective or impractical.	SURGIFLO™ Hemostatic Matrix, mixed with sterile saline or thrombin solution, is indicated in surgical procedures (other than ophthalmic) as an adjunct to hemostasis when control of bleeding by ligature or other conventional methods is ineffective or impractical.	The SURGIFLO™ Endoscopic Applicator is intended for use in delivering hemostatic agents to bleeding surgical sites through a 5 mm trocar or larger.
Size	8 mL	8 mL	34cm
Hemostatic Agent & Accessory	Hemostatic Agent	Hemostatic Agent	Accessory
Absorption Time	4-6 weeks	4-6 weeks	4-6 weeks
Storage Requirements	Stored dry at controlled room temperature 36°-77°F (2°-25°C).	Stored dry at controlled room temperature 36°-77°F (2°-25°C).	Store at controlled room temperature 59°-86°F (15°-30°C)
Preparation Time	Ready to use in 30 seconds or less once liquid thrombin is delivered to sterile field	Ready to use in 30 seconds or less once liquid thrombin or saline is delivered to sterile field	Ready out of package
Shelf Life	2 years	2 years	5 years
Material/Composition	Porcine Gelatin and Human Thrombin	Porcine Gelatin	Stainless Steel Cannula and Plastic Obturator
Mechanism of Action	SURGIFLO™ Hemostatic Matrix with thrombin provides a matrix for platelet adherence, accelerating the formation of the platelet plug and aids in fibrin clot formation	SURGIFLO™ Hemostatic Matrix without thrombin provides a matrix for platelet adherence, accelerating the formation of the platelet plug	--
NDC/Distributor Code	63713-461 (Lyophilized)	63713-0029-91	63713-MS19-95
Qty./Box	6	6	6